

HANDS-FREE HVE

# Forest

DENTALEZ

AEROSOL EXTRACTION WITHOUT THE DISTRACTION.™

COMPATIBLE WITH ALL DENTALEZ AND FOREST™ PATIENT CHAIRS, AND MOST CHAIRS FROM OTHER MANUFACTURERS


For compatibility with a non-DENTALEZ chair, please reference the below diagrams showing the interface between the headrest glide bar and the HVE holder (FIG. 1), and a headrest fit guide (FIG. 2) that can be placed over your headrest glide bar to verify fit.

HEADREST GLIDE BAR DIMENSIONS (W X T)

2.2" x 0.24" (55mm x 6mm) OR

1.6" x 0.4" (40mm x 10mm)

! NOTE: Emailed, faxed, or copied templates may jeopardize precise full scale measurements. For best results please print in actual size.

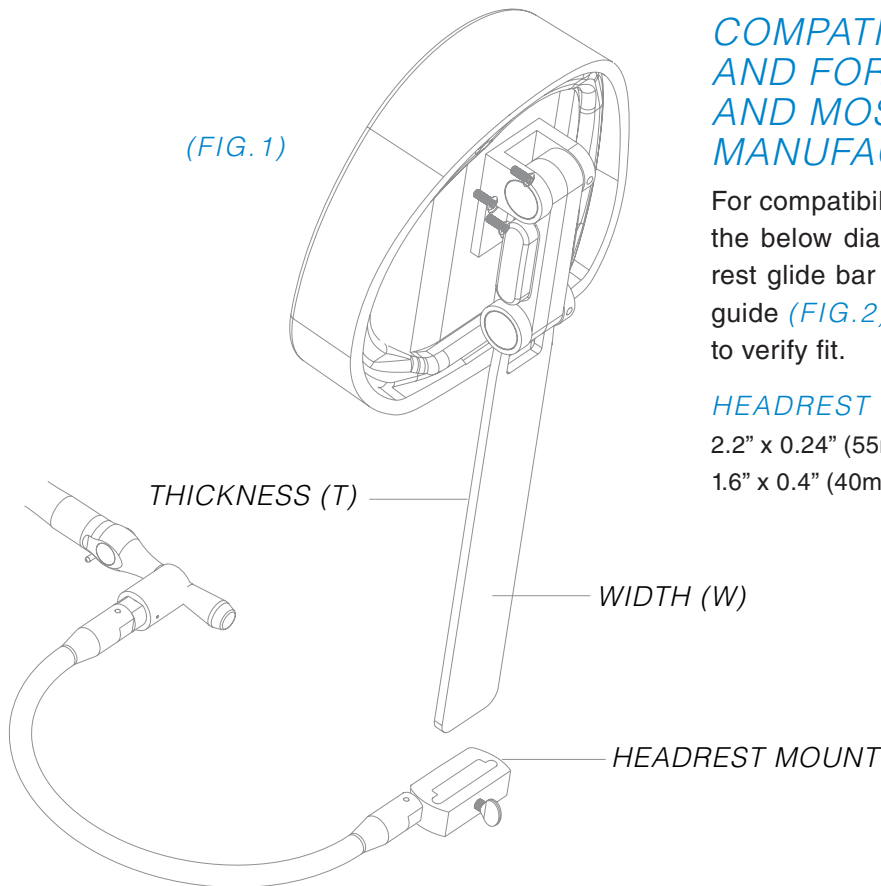

CUT & FIT TEMPLATE BELOW ON YOUR HEADREST GLIDE BAR

HEADREST FIT GUIDE (FIG. 2)

